


Sticker Printing


Q: Where can I find the stickers?

You can find stickers for all completed cases in your Personal Account.

- 1) Please open the case you want to print the stickers for

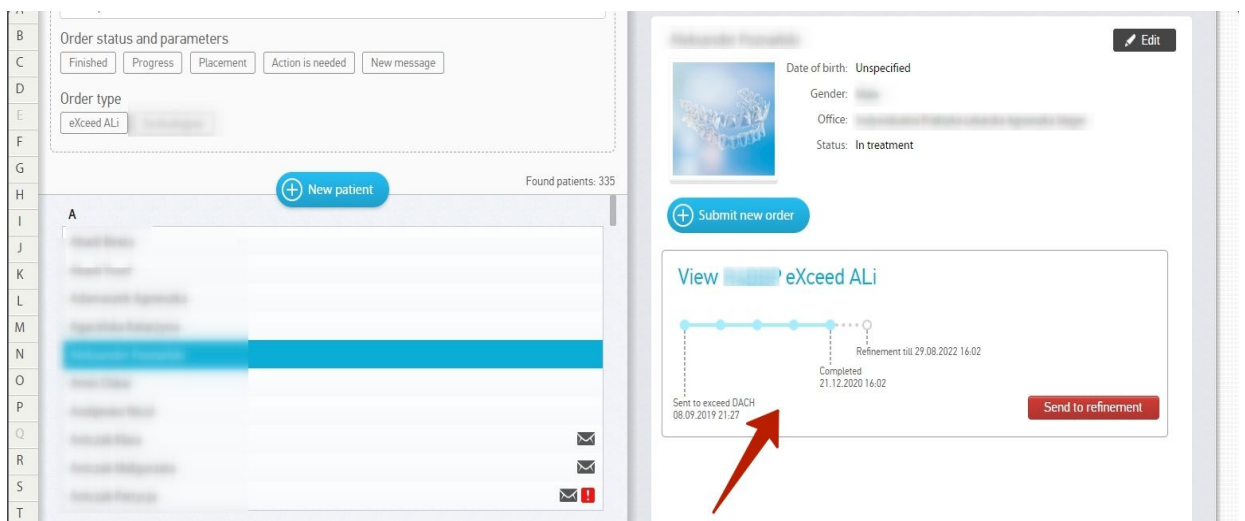

- 2) Find the approved setup and click on the “piece of paper” icon

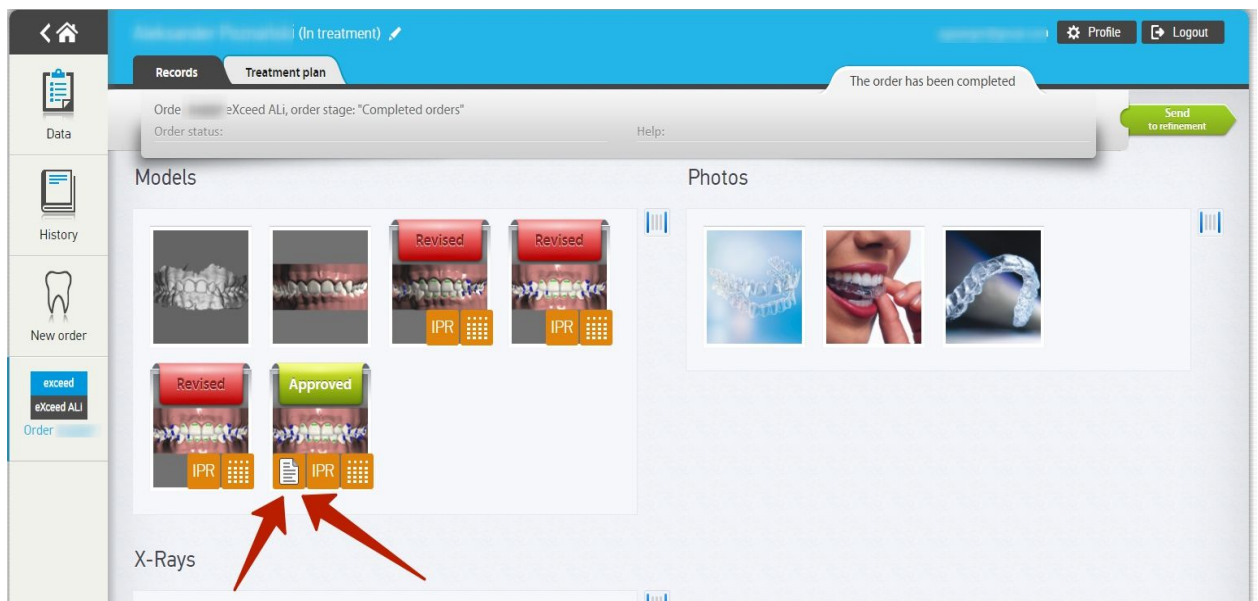

- 3) You'll be able to print the stickers or save them in .pdf by clicking the right mouse button and selecting "Print"

ID: [REDACTED]
Doctor: [REDACTED]
Patient: [REDACTED]

Upper arch 29 aligners
Lower arch 31 aligners

Total IPR required:
11][21 - 0.4mm

04.02.2021

ID: [REDACTED]
Doctor: [REDACTED]
Patient: [REDACTED]

Aligner upper: 1 from 29
Aligner lower: 1 from 31

04.02.2021

Q: What printers are recommended?

You can use any printer of your choice. We recommend using **EPSON TM-C3500**
In eXceed we use an Epson printer and its quality is time proven for intense production.



Q: What settings should I use (mm*mm)?

We recommend the settings that eXceed uses, 80x72mm:

