




3Shape Trios Bridge

05.2017

# 1.1 – Connecting Trios with eXceed

3shape 

English

3shapedental.com

## 3Shape Communicate

### Log In

Please enter your email address and password. [Register](#) if you don't have an account.

UserEmail

Password

Remember me?

[I forgot my password](#)

Copyright © 3Shape A/S 2011 • [info@3shape.com](mailto:info@3shape.com) 3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620

Log in to your 3Shape Communicate account

## 1.2 – Connecting Trios with eXceed

The screenshot displays the eXceed software interface. The top navigation bar includes a 'Settings' tab and a 'Configure' button (gear icon) which is highlighted with a yellow border. The left sidebar contains a menu with items: General, System, Operator, Connection, Lab Connections (highlighted in green), Order, Scan, and Messages. The main content area is titled 'Lab Connections' and features the following fields and controls:

- External Connection Method:** A dropdown menu currently showing '3Shape Communic'.
- Communicate account:** A button labeled 'Create Account'.
- Communicate login (email):** A text input field containing 'heidi\_schultz@yahoo.com', highlighted with a yellow border.
- Communicate password:** A password input field with masked characters (dots), also highlighted with a yellow border.
- Test Account:** A button located below the password field, highlighted with a yellow border.
- Laboratories:** A section containing 'Add Lab' and 'Refresh' buttons, and a list item 'Practice lab'.


In “Configure/Settings”, test account connectivity

## 1.3 – Connecting Trios with eXceed

The screenshot displays the eXceed software interface. On the left is a vertical navigation menu under the heading "Settings". The menu items are: General, System, Operator, Connection, Lab Connections (highlighted in green), Order, Scan, and Messages. The main content area is titled "Lab Connections". It features a dropdown menu for "External Connection Method" set to "3Shape Communic". Below this are three input fields: "Communicate account" with a "Create Account" button, "Communicate login (email)" with the text "heidi\_schultz@yahoo.com", and "Communicate password" with a masked password field and a "Test Account" button. At the bottom of the main area, under the heading "Laboratories", there are two buttons: "Add Lab" (highlighted with a yellow box) and "Refresh". Below these buttons, a list shows "3Shape Official Demo Lab".

In "Lab Connections", click "Add Lab"

# 1.4 – Connecting Trios with eXceed

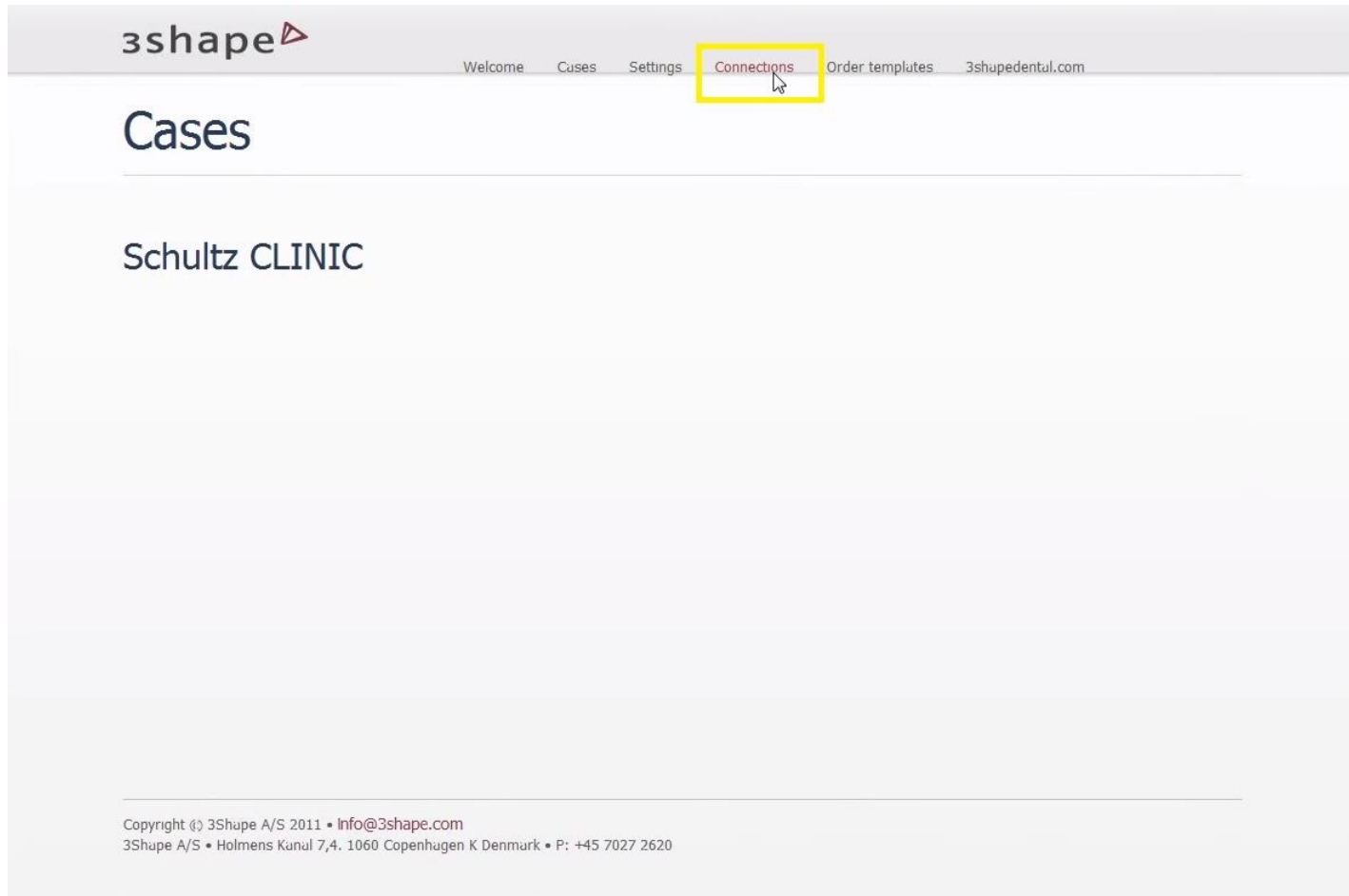

Click “Connections”

# 1.5 – Connecting Trios with eXceed

## Connected Sites

Name

Status

Add Connection

Click “Add Connection”

# 1.6 – Connecting Trios with eXceed

The screenshot shows the 3shape website interface. At the top left is the 3shape logo. A navigation bar contains links for Welcome, Cases, Settings, Connections, Order templates, and 3shapedental.com. The main heading is "Connect to other Dental Clinics and Labs". Below this, there is a form with two rows. The first row is for "North America" and contains an email input field with "info@exceed-ortho.com" and a "Find" button. The second row is for "Europe" and contains an email input field with "tech@exceed-ortho.com" and a "Find" button. Both email fields and "Find" buttons are highlighted with yellow boxes. At the bottom of the page, there is a footer with copyright information: "Copyright © 3Shape A/S 2011 • Info@3shape.com" and "3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620".

Enter [info@exceed-ortho.com](mailto:info@exceed-ortho.com) for North-America **OR** [tech@exceed-ortho.com](mailto:tech@exceed-ortho.com) for Europe and click “Find”

# 1.7 – Connecting Trios with eXceed

The screenshot shows the 3shape website interface. At the top, the 3shape logo is on the left, and navigation links for 'Welcome', 'Cases', 'Settings', 'Connections', 'Order templates', and '3shapedental.com' are on the right. The main heading is 'Connect to other Dental Clinics and Labs'. Below this, there is a form with the following elements:

- A text input field containing 'info@exceed-ortho.com OR tech@exceed-ortho.com', highlighted with a yellow box.
- A 'Find' button to the right of the input field.
- A 'Result:' label above a text box containing 'eXceed', also highlighted with a yellow box.
- A 'Message to' dropdown menu set to 'LAB (You can customize the message)'.
- A text area containing the message: 'We would like to connect to you on 3Shape Communicate.'
- A 'Connect' button at the bottom right of the message area, highlighted with a yellow box.


At the bottom of the page, there is a footer with the following text:

Copyright (c) 3Shape A/S 2011 • Info@3shape.com  
3Shape A/S • Holmens Kanal 7,4. 1060 Copenhagen K Denmark • P: +45 7027 2620

Verify that the system identifies eXceed and click “Connect” to complete the bridge. Exceed will send you a confirmation once the bridge has been established successfully




## 2.1 – Submitting a Trios scan to eXceed

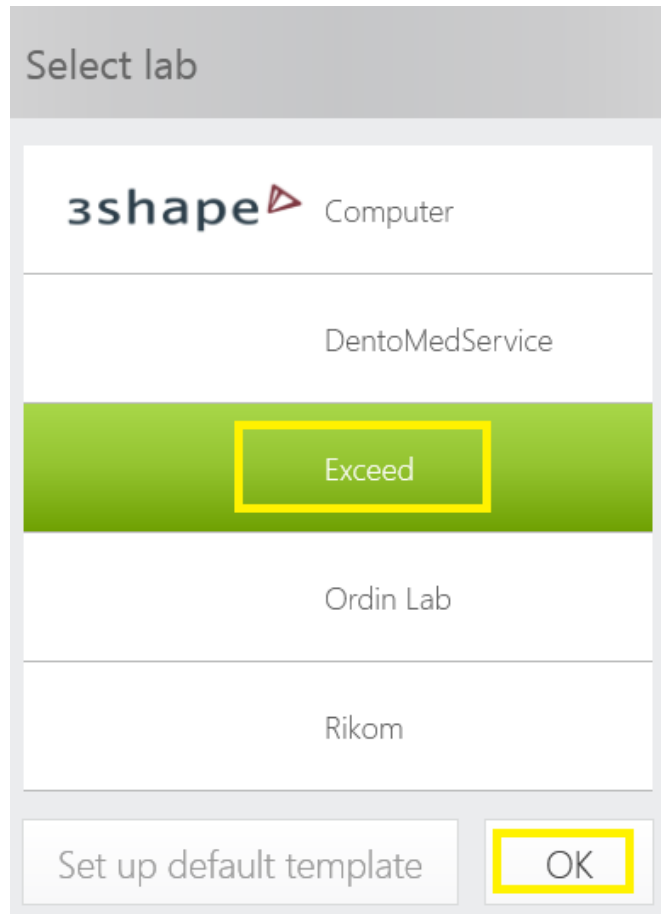


In the Trios interface, choose “eXceed” from the “Select lab” menu

## 2.2 – Submitting a Trios scan to eXceed

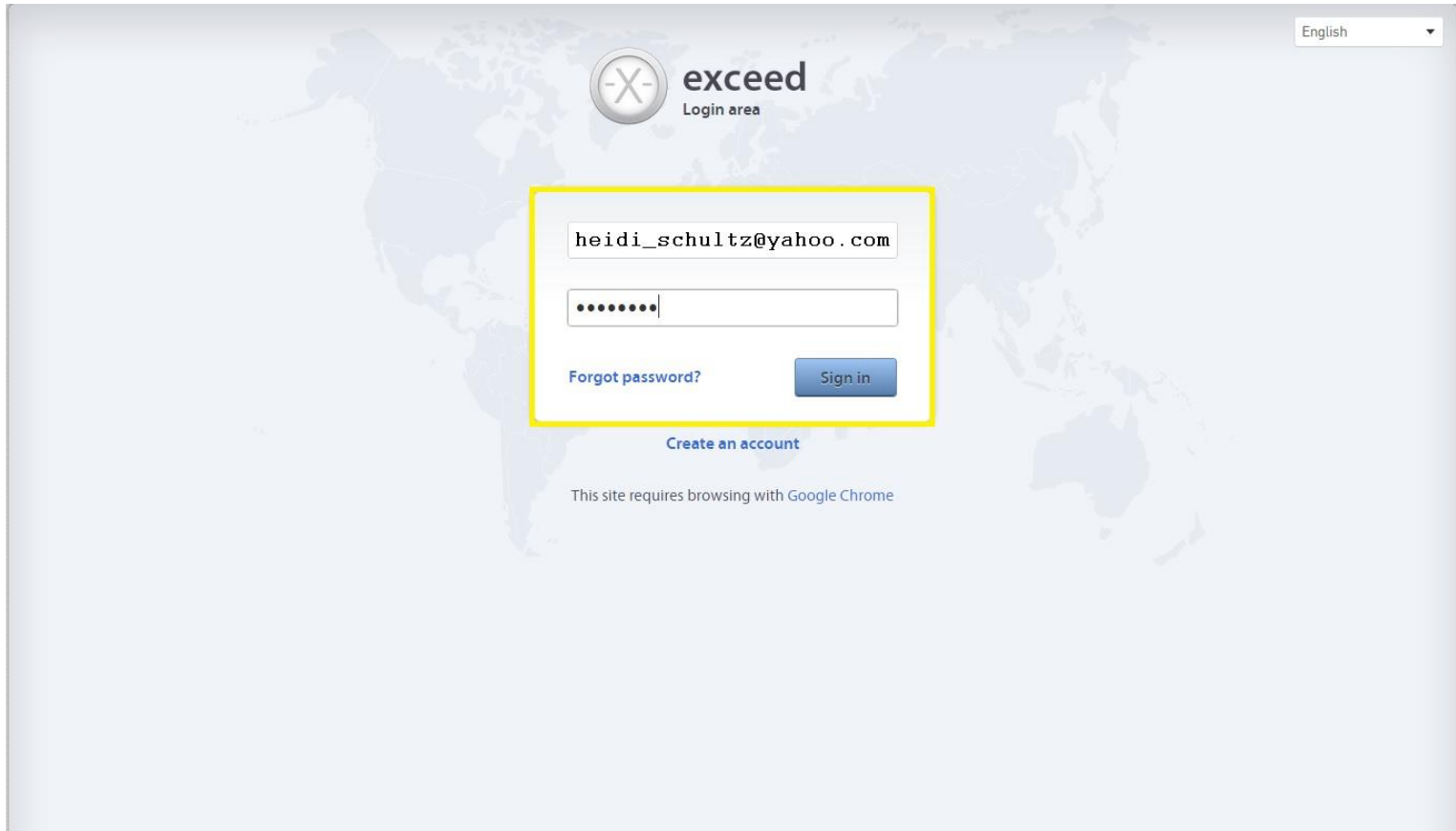

English

 **exceed**  
Login area

[Forgot password?](#)

[Create an account](#)

This site requires browsing with Google Chrome

Log in to My eXceed

## 2.3 – Submitting a Trios scan to eXceed

Altman April (In treatment) Profile Logout

Records Prescription Delivery Checkout

Place a new order Continue

**Models** Add

Patient ID: 12345678 + hit "Enter"

Select:  0 Digital model  0 FX  Retrieve model from scanner

Add to order

Select

Enable access to models from iTero/Trios intra-oral scanners

Records

History

New order

exceed  
exceed TX  
Order EMR35

exceed  
exceed RX  
Order ZKEN6

Hit "Select", type in Trios scan ID and hit "Enter" on your keyboard